



Oxivir® Tb Wipes

General Virucide, Bactericide, Tuberculocide, Fungicide, Sanitizer

A ready-to-use pre-wetted disinfectant cleaner wipe based on proprietary AHP® technology to deliver fast, effective cleaning performance.

Features & Benefits

- One-step cleaner disinfectant powered by AHP® technology
- Effective against key pathogens – TB, MRSA, Norovirus and VRE
- Kills HBV, HCV and HIV in 1 minute. Tuberculocidal in 5 minutes and Fungicidal in 10 minutes
- No VOCs, no added fragrance and no NPEs. Environmentally responsible – active ingredient breaks down to oxygen and water
- Compatible with most hard, non-porous surfaces
- Convenient – pre-wetted wipes make even the toughest job easier

Applications

- Clean and disinfect high-touch surfaces in just ONE Minute
- Non-irritating to eyes and skin





Oxivir® Tb Wipes

General Virucide, Bactericide, Tuberculocide, Fungicide, Sanitizer

Use Instructions

As a One-Step Cleaner Disinfectant

- Pre-clean heavily soiled areas
- Apply product by coarse trigger spray or disposable wipe to hard, non-porous inanimate surfaces
- All surfaces must remain visibly wet for 1 minute
- Use 5-minute contact time for TB and a 10-minute contact time for fungi. Air Dry, wipe surfaces to dry and remove any residue, or rinse with potable water as necessary.

Note: Rinsing is not necessary unless floors are to be coated with finish or restorer. All food contact surfaces must be rinsed with potable water. Do not use on glassware, utensils, or dishes.

Prior to first use on a particular surface, it is recommended to test in a small, discreet area if there is concern of incompatibility, or contact a Diversey representative.

Technical data	Oxivir® Tb Wipes
EPA Reg. No.	70627-60 (Canisters, Buckets, Refills) 74559-3 (Soft Packs)
Color/Form	Colorless, wipe
pH	3
Scent	Characteristic of ingredients
Shelf Life	2 years - 6 X 7 in Wipes (Canister) 3 years - 7 X 8 in Wipes (Canister), 7.48 x 10 in Wipes (Soft Packs), 11X12 in Wipes (Buckets & Refills)

Product	Pack size	Dilution	Product code
Oxivir® Tb Wipes	12 x 160 count 6 x 7 in. Wipes / 15.24 x 17.78 cm Canisters	Ready-to-use	4599516
Oxivir® Tb Wipes	12 x 60 count 7 x 8 in. Wipes / 17.8 x 20.3 cm Canisters	Ready-to-use	5388471
Oxivir® Tb Wipes	4 x 160 count 11 x 12 in. Wipes / 27.9 x 30.5 cm Buckets	Ready-to-use	5627427
Oxivir® Tb Wipes	4 x 160 count 11 x 12 in. Wipes / 27.9 x 30.5 cm Refills	Ready-to-use	100823906
Oxivir® Tb Wipes	120 X 10 count 7.48 X 10 in. Wipes / 18.99 X 25.4 cm Soft Packs	Ready-to-use	100949406
Oxivir® Tb Wipes	6 x 1 Unit (Wall Bracket for use with 60 count Canisters)		9P700827
Oxivir® Tb Wipes	1 Double Unit Non-Locking Stand		9P700798
Oxivir® Tb Wipes	1 Double Unit Locking Stand		9P700797
Oxivir® Tb Wipes	6 x 1 Unit Wire Bracket Locking on Plastic Wall Mount (for use with 160 count Canisters 6"X7")		9P700799
Oxivir® Tb Wipes	6 x 1 Unit Wire Bracket Non-Locking on Plastic Wall Mount (for use with 160 count Canisters 6"X7")		9P700800
Oxivir® Tb Wipes	4 x 1 Unit Large Bucket Wall Mount Bracket (for use with 160 count Buckets)		9P700829

Safe handling

Please make sure your employees read and understand the product label and Safety Data Sheet before using this product. The label contains directions for use; and both the label and SDS contain hazard warnings, precautionary statements and first aid procedures. SDS are available online at www.diversey.com or by calling 888.352.2249. Improper use or dilution may result in damage to surfaces and may result in health and physical hazards that match those of the concentrate. Please refer to the Diversey HazMat Library, only available through Internet Explorer, <http://naextranet.diversey.com/dot/>, for up-to-date shipping information.

AHP and Design® are registered trademarks of Virox Technologies, Inc. used under license.

www.diversey.com

© 2018 Diversey, Inc. All Rights Reserved. PSS729 en (18/303)