

Oxivir® 1 Wipes

One-step Ready-to-use Hospital Disinfectant Cleaner Wipe

Next generation ready-to-use, pre-wetted, one-step disinfectant cleaner wipe, based on proprietary AHP® technology. Oxivir® 1 Wipes are formulated to deliver broad spectrum disinfection in ONE minute or less, with 75 total claims.

Features & Benefits

- Broad spectrum, multi-surface cleaner/disinfectant and sanitizer. Harnesses the cleaning power of hydrogen peroxide. Specially designed to penetrate the toughest soils
- Fast, Effective, Responsible and Sustainable formulation. Environmentallyresponsible with no VOC's, no added fragrance, no NPE's. Fast-acting, convenient formula, suitable for daily use, and contact time of ONE minute or less
- Formulated with patented AHP® technology containing no phosphates, quats or phenols. The active ingredient breaks down to water and oxygen after use
- Short contact times of 1 minute or less Virucidal, 1 minute Bactericidal,
 Tuberculocidal and Fungicidal and 10-second non-food contact sanitizing
- Meets the lowest EPA toxicity category in all 6 toxicity studies, making it a
 responsible choice for users, around patients and on most common healthcare
 surfaces. Cleans, disinfects and deodorizes in one labor-saving step

Applications

 This product can be used in multiple areas including operating rooms, longterm care facilities, medical facilities, animal treatment facilities, cruise ships, hotels, public buildings, public restrooms, recreational facilities, retail businesses and schools (colleges and universities), and is compatible with most hard, non-porous surfaces







Oxivir® 1 Wipes

One-step Ready-to-use Hospital Disinfectant Cleaner Wipe

Use Instructions

- Pre-clean heavily soiled areas
- Wipe hard, non-porous environmental surfaces with towelette
- Allow surface to remain wet for 1 minute. (For certain listed viruses, allow to remain wet for 30 seconds).
- Air Dry, wipe surfaces to dry and remove any residue, or rinse with potable water as necessary

Note: All food contact surfaces must be rinsed with potable water. Do not use on glassware, utensils, or dishes.

Oxivir $^{\circ}$ 1 is compatible with most common healthcare surfaces. Prior to first use on a particular surface, it is recommended to test in a small, discreet area if there is concern of incompatibility, or contact a Diversey representative.

Technical data	Oxivir [®] 1 Wipes	
EPA Reg. No	70627-77	
Color/Form	Colorless, wipe	
рН	1.7-2.5	
Scent	Characteristic of ingredients	
Shelf Life	2 Years	

Product	Pack size	Dilution	Product code
Oxivir [®] 1 Wipes	12 x 60 count wipes 7 in. x 8 in. /	Ready-to-use	100850922
	17.8 cm x 20.3 cm Canisters		
Oxivir [®] 1 Wipes	12 x 160 count wipes 6 in. x 7 in. /	Ready-to-use	100850923
	15.24 cm x 17.78 cm Canisters		
Oxivir [®] 1 Wipes	12 x 60 count wipes 10 in. x 10 in. /	Ready-to-use	100962573
	25.4 cm x 25.4 cm Canisters		
Oxivir® 1 Wipes	4 x 160 count wipes 11 in. x 12 in. /	Ready-to-use	100850924
	27.9 cm x 30.5 cm Buckets		
Oxivir® 1 Wipes	4 x 160 count wipes 11 in. x 12 in. /	Ready-to-use	100850925
	27.9 cm x 30.5 cm Refills		_

Safe handling

Please make sure your employees read and understand the product label and Safety Data Sheet before using this product. The label contains directions for use; and both the label and SDS contain hazard warnings, precautionary statements and first aid procedures. SDS are available online at www.diversey.com or by calling 888.352.2249. Improper use or dilution may result in damage to surfaces and may result in health and physical hazards that match those of the concentrate. Please refer to the Diversey HazMat Library, only available through Internet Explorer, http://naextranet.diversey.com/dot/, for up-to-date shipping information.

AHP and Design® are registered trademarks of Virox Technologies, Inc. used under license.